[EN] DROPTonic® 5% is a hypertonic ophthalmic solution for the treatment of corneal oedema occurring as a result of mechanical, toxic, dystrophic or inflammatory damage. DROPTonic® 5% makes the tears hypertonic, drawing up the excess water from the corneal tissue by osmosis, reducing thus the oedematous state. **INDICATIONS:** DROPTonic® 5% is indicated for: Visual discomfort, burning and eye irritation caused by corneal oedema. In case of abnormal fluid accumulation caused by corneal oedema, with or without inflammation or blunt trauma after refractive or ophthalmic surgery, cataracts, transplants, retinal detachments, corneal transplants, corneal endothelial decompensation and in all dystrophic, mechanical, toxic and inflammatory corneal decompensations. **INSTRUCTIONS FOR USE:** 1. Remove the cap. 2. Gently squeeze the bottle and apply 2 drops in each eye; repeat as required. Immediately close the container after use: the solution can be used for up to 3 months after the first opening. **COMPOSITION:** Riboflavin sodium phosphate (0.000075%), Sodium chloride (5%), Vitamin E TPGS, Sodium Hyaluronate, L-Proline, L-Glycine, L-Leucine, L-Lysine, Sodium phosphate dibasic dodecahydrate, Sodium phosphate monobasic monohydrate, purified water. Preservative free. **WARNINGS AND PRECAUTIONS:** The product is intended for external ophthalmic use only. The product is intended for single patient use. Do not use in case of known intolerance or hypersensitivity to one of its components. Do not use in case of damaged or not tightly closed package before the first opening. Do not use the product after the expiry date indicated on the package. In case of problems during the use, discontinue the treatment and consult a doctor. During the product application, do not touch the eye or any other surface with the dispenser tip. The expiry date refers to an unopened and correctly stored product. Before use, carefully read the instructions for use. Keep away from children. Do not dispose into the environment after use. The solution may stain surfaces and fabrics. In rare cases, corneal microcalcifications have been observed, in patients with a history of corneal disease treated with ophthalmic preparations containing phosphates. In this case it is recommended to use the product under medical supervision. Transparent straw yellow solution. In case of contact with skin or clothes, is easy to remove. It is possible to report adverse events to the legal manufacturer via the email address: info@iromedgroup.com. By reporting adverse events, you can contribute to the safety of this product. Any severe event should be notified to the legal manufacturer and to the Competent Authority of the country in which this product has been used. **PACKAGING:** 10 ml multidose dispenser, preservative free. **STORAGE:** Immediately close the dispenser after use: the remaining product can be used up to 3 months after the first opening. Keep the product away from heat sources and in any case at a temperature below 25°C. **EXPIRY:** With an undamaged package, do not use after the expiry date indicated both on the box and the dispenser. Do not separate the dispenser from the box and the leaflet, to keep all the information for a correct use in the same place.

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TRB CHEMEDICA SAS
TRB CHEMEDICA (Austria) GmbH